MGH00107

Valid from / en cours de validité depuis le / en espera de validación desde el / in corso di validità dal / gültig seit dem

15/03/2021

**Product Sheet (EN) 2**

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**Eigenschaften (DE) 10**

PRODUCT SHEET

Valid from 15/03/2021

<b>IDENTIFICATION OF THE MEDICAL DEVICE</b>	Type	Rechargeable battery	 
	Commercial designation	Pack Bio NIMH 12V 1400mAh ATMOS ATMOVAC	
	Reference	MGH00107	
	EAN	3660766502070	
	Brand	NX	
	Compatible / Original battery	Compatible	
	Packaging	Unitary	

RECOMMENDED USAGE

Follow the instructions and recommendations specific to each model, using the technical instructions and document resources from the devices in which the battery is used

Brands	Equipment	Models	PN
Atmos	Aspirateur de mucosites	Tracheal ATMOVAC	

<b>Identification</b>	<b>GENERAL TECHNICAL CHARACTERISTICS</b>	Chemistry	Ni-Mh
		Type	VH AA
		IEC designation	10 HRM15/49
		Rated voltage	12V
		Nominal capacity	1,4A
		Internal resistance $\Omega$	<190m $\Omega$

The voltage and the actual capacity in use can be affected by several factors, especially the temperature, the discharge current, the pack's history (ex:use, storage), etc

<b>ELECTRICAL CHARACTERISTICS</b>	<b>CHARGE</b>	Maximum charging voltage	17V
		Standard charging current (16h)	140mA
		Fast charging current (2,5h)	1A
	<b>DISCHARGE</b>	Range of operating voltage	10V at 17V
		Min tension in discharge	10V
		Max discharge current	1,4A
		Lifespan 80% DOD (0,5 C)	500 Cycles
	<b>MAINTENANCE</b>	Frequency of maintenance charges at 20°C	3Months
	<b>CONTROL ELECTRONICS</b>	Electrical protection	Yes
		Low voltage power cut	No
		High voltage power cut	No
		Max power cut voltage	Yes

These devices not only designed to protect the pack in case of an equipment failure. They must not be used to control the discharge. The protection circuits have a response time of a few milliseconds.

<b>MECHANICAL CHARACTERISTICS</b>	Dimensions (+/- 2mm)	Length	70,8mm
		Width	28,5mm
		Depth	50mm
	Weight (+/- 5g)		260Gr
	Mechanical protection		Sleeve
	Wire length (+/- 10mm)		
	Terminal		Faston

<b>CONDITIONS OF USE, STORAGE, AND TRANSPORT</b>	<b>CONDITIONS OF USE</b>	Charging temperature	0 at +40°C
		Discharge temperature	-15 at +40°C
	<b>CONDITIONS OF STORAGE</b>	Storage temperature	+5 at +25°C
		Level of humidity	65,00 %
		Max storage time	2 Years
	<b>TRANSPORT</b>	UN code	Class 9
		ADR/RID classification	Class 9
		IMDG classification	Class 9
		IATA classification	Class 9

<b>INSTRUCTIONS</b>	<b>COMMISSIONING</b>	<ul style="list-style-type: none"> <li>• Check the batteries and the connectors: wires not damaged, battery not swollen, burnt smell, oxidation of the connectors, leak...</li> <li>• Respect the polarity</li> <li>• Do a full charge with the adequate charger before the first use</li> </ul>
	<b>CHARGE</b>	<ul style="list-style-type: none"> <li>• Use an adequate charger</li> <li>• The battery is warmer during the charge: during the first charge, check that the battery's temperature stays in the temperature operating ranges.</li> <li>• In case of an abnormal heating, stop the charge by unplugging the charger within the realms of possibility, remove the battery from the equipment de l'équipement, have the equipment, the battery and the charger checked by a technician.</li> </ul>
	<b>CASE OF NON-WATERPROOF BATTERIES</b>	<ul style="list-style-type: none"> <li>• It is normal to observe a release of gas during the charge and use. Do not smoke. Place in suitable premises.</li> <li>• Open batteries need regular maintenance carried out by a qualified technician.</li> </ul>
	<b>CASE OF LITHIUM ION BATTERIES</b>	There is a fire hazard with lithium ion batteries in the following cases: overload, short circuit, charge and use outside the voltage and temperature ranges.
	<b>WARNINGS</b>	<ul style="list-style-type: none"> <li>• Read the instructions of your device.</li> <li>• Only use in compatible devices.</li> <li>• Respect the load and storage conditions.</li> <li>• Do not use if the battery is damaged, do not burn, do not pierce, do not dismantle or modify. The protection circuits protect the battery and the equipment: do not deactivate them.</li> </ul>

<b>EXPLANATION OF SYMBOLS</b>	<b>LABELLING</b>	
		Catalogue reference
		Lot number
		manufacturer's address
		To recycle in a suitable salvage and recycling structure
		Read the product sheet and the instruction manual

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